

## 514/B Carry Sheets



To join the two halves for use as a patient stretcher, simply line up the central loops and insert the plastic spine.



Once the spine is in place connect the metal safety catches on each end.



The metal handles are inserted through the small webbing loops on each side of the carry sheet (as shown) and should pass through four (4) loops. The black plastic rings on each handle are positioned to restrict the handle from sliding through the webbing loops when in use.



Once in place the metal handles give ample room for large hands to get a good grip. The handles are fluted for additional grip.



The end handles may be deployed and used to drag a patient on the carry sheet. The handle is normally stowed away via velcro.

### Specifications

- 160kg S.W.L
- L1670mm x W610mm
- Weight 3.6kg
- 12 handles inc. handles.

Note :  
514 Carry sheet does **NOT** include aluminium handles  
514B **DOES** include handles

### Optional Accessories



519C Carry Bag

### Operations...

1. From a kneeling position operators positions themselves close to the patient who is then rolled onto their side. The carry sheet is then placed under the patient who is then rolled back onto the sheet. A roll to the opposite side is usually required to access the handles .
2. Lifters should be placed evenly on each side of the patient before lifting.
3. The carry sheet is then lifted & loaded onto a stretcher lowered to half height.
4. For comfort hands can grip anywhere along the handle. When walking the rear hand should face palm down & the forward hand palm up.
5. Because the grips are fixed below the patient, operators must start the lift from a full squat position. Ensure a wide stance to prevent overbalancing.
6. **To prevent the sheet splitting, ensure the plastic & metal security clips are fastened.**

### CLEANING INSTRUCTIONS

Clean all parts with warm soapy water or mild anti-bacterial solution. Ensure thorough rinsing with clean water before air drying

DHS Emergency  
321 Milperra Rd Milperra, NSW  
Ph 612 8724 6833  
Fax 612 9774 2922  
www.dhsemergency.com

54 Armada Place, Banyo, Qld  
Ph. 617 3267 1730  
Fax 617 3267 1740

66 Allied Dr, Tullamarine, VIC  
Ph. 613 9330 0055  
Fax 613 9330 0155